

EST.
2015



SHOE
MFG

ANODYNE


PREMIUM QUALITY

Anodyne Scanner App Instructions




Once you've downloaded the Anodyne App and attached your Structure camera, you're ready to go! Below are step by step instructions on how to properly submit an order.


1. Click on the Anodyne App Icon to launch the scanner application.

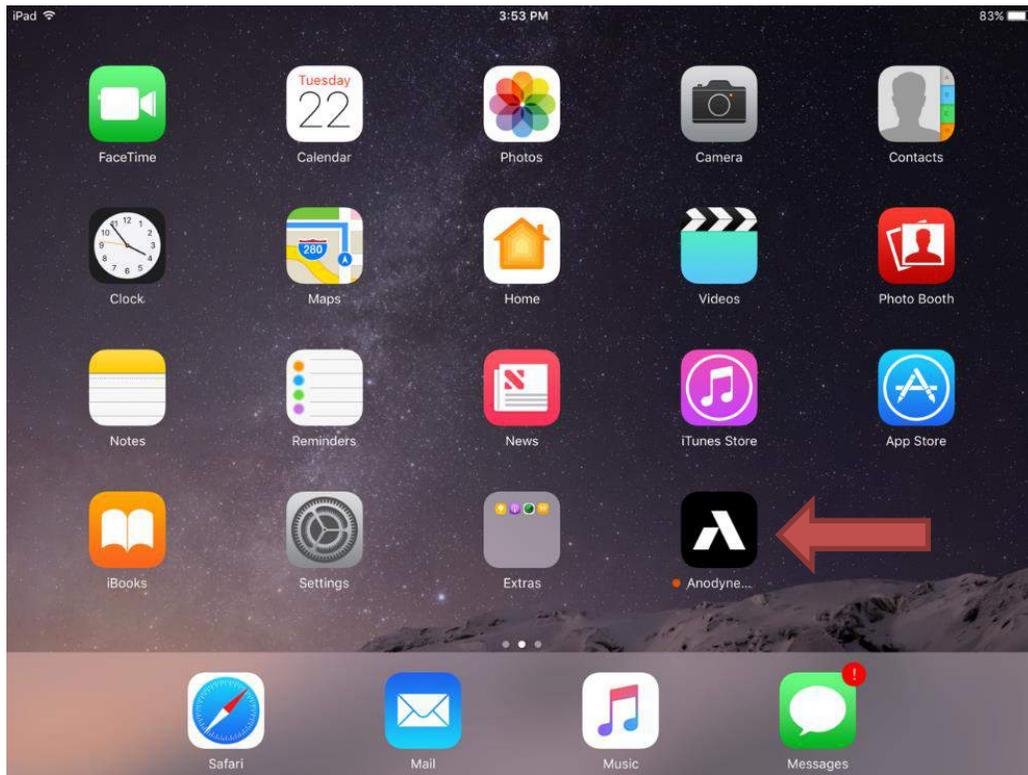


2. Select the "Settings" button located on the bottom right of the screen.

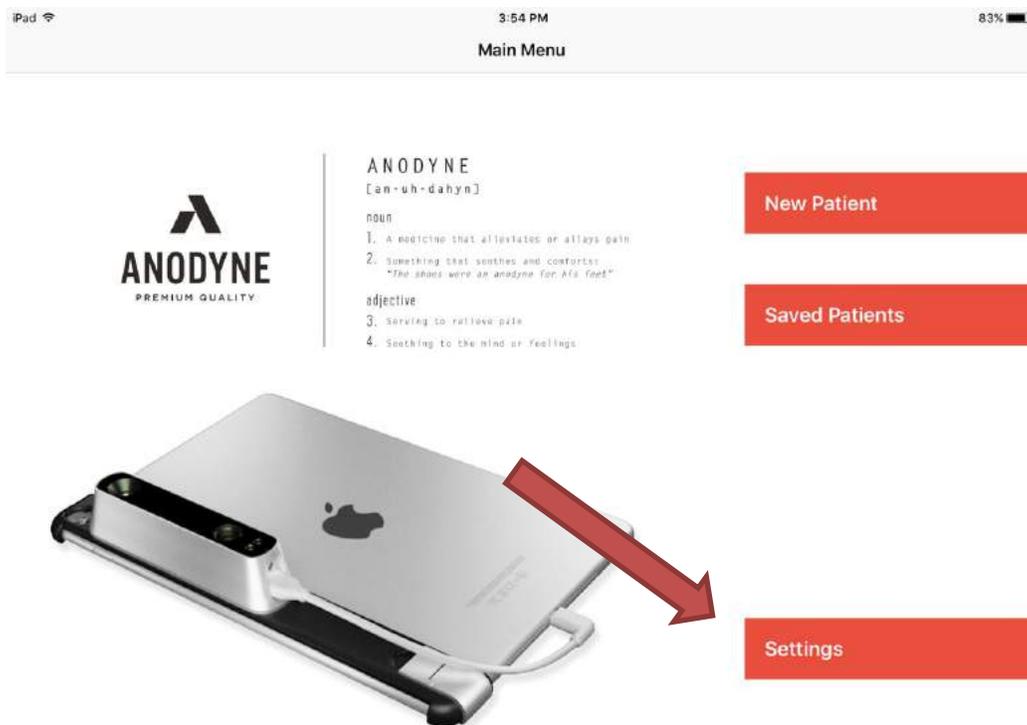

3. Now that you're in the "Settings" menu, enter your account information in all of the different fields. Click "Save". Then go back to the main menu by clicking "Main Menu" in the top left corner. You'll only have to do this once.

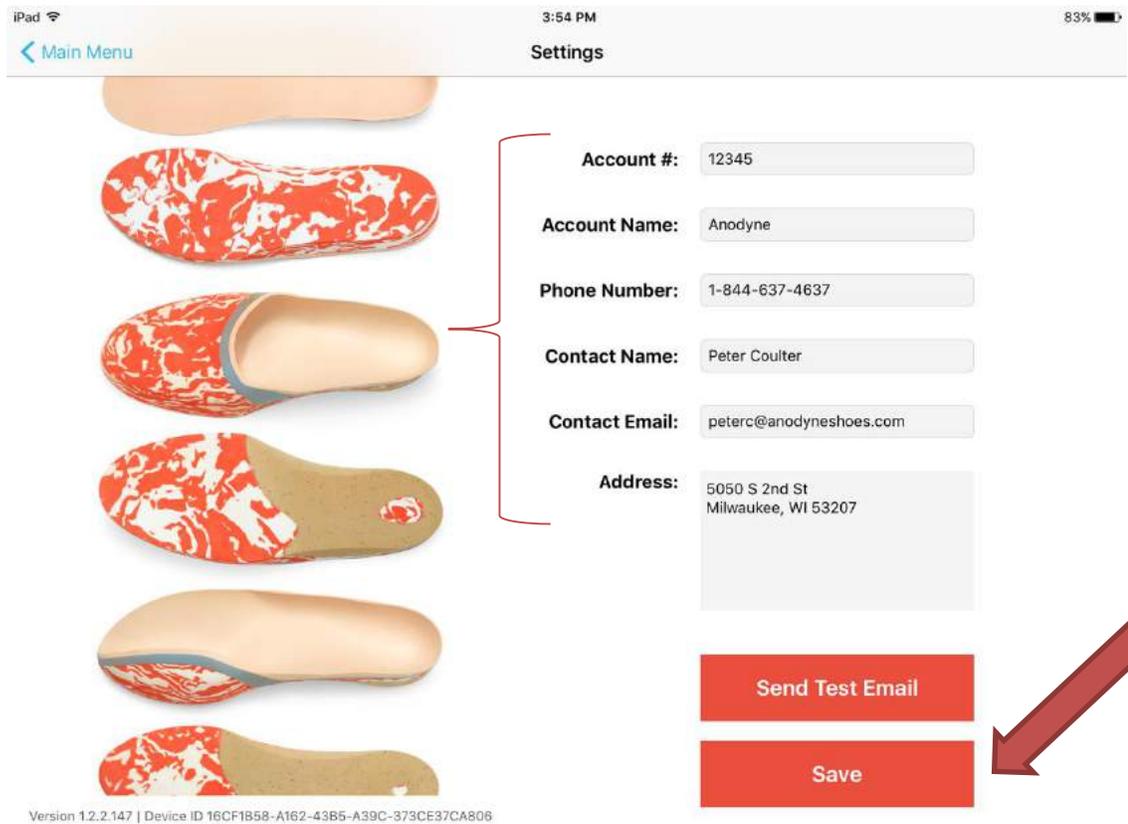

4. You're now ready to scan a patient. Select the "New Patient" button located on the top right of the screen.

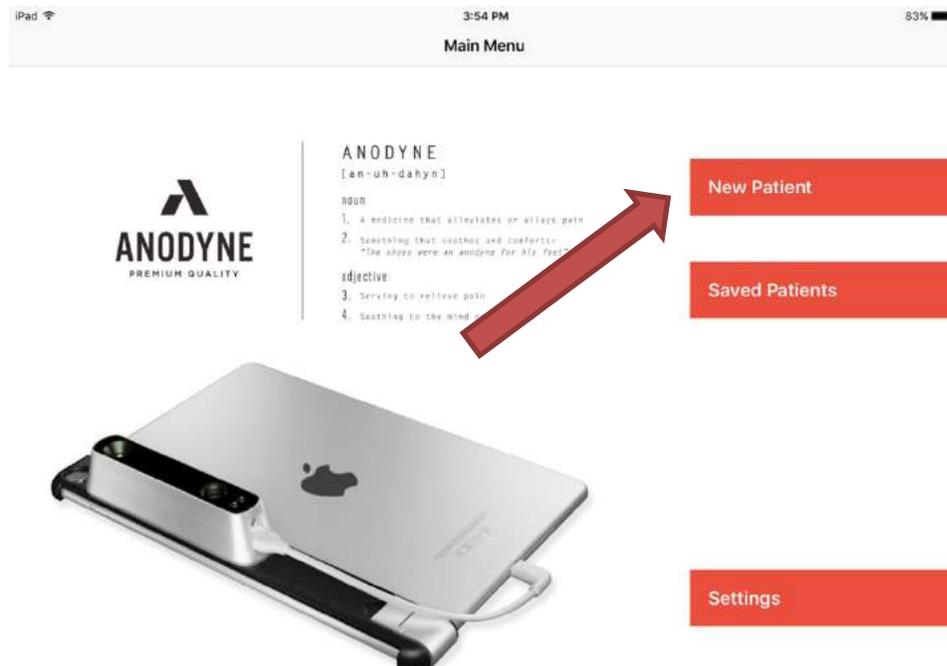

5. This is the “Patient Form” and is just a digital version of our standard order form. At the very minimum, fill out all of the fields that are highlighted in red. Those fields are required to be filled before you can move on to the next step. The account information in the top right corner will automatically populate, based on the information that you entered in the “Settings” screen.

iPad 3:54 PM 83%

[Main Menu](#) Patient Form

5050 South 2nd Street
Milwaukee, WI 53207
1-844-637-4637
www.anodyneshoes.com

ANODYNE

Account No: 12345
Account Name: Anodyne
Date: 11/22/16
P.O. No: _____

Patient Name: _____ Height: _____ Weight: _____

Activity Level: Low Moderate High Shoe Order Below Inserts Only Order

Insert Quantity (pairs): 1 2 3 Toe Filler: Left Right

Men's: Size: 7.5 8 8.5 9 9.5 10 10.5 11 11.5 12 13 14
Width: M W XW

Women's: Size: 5.5 6 6.5 7 7.5 8 8.5 9 9.5 10 11 12
Width: M W XW

Arch Height: Same as scan
Lower than scan
Higher than scan

Off-Load Metatarsal Heads:
Right: 1 2 3 4 5
Left: 1 2 3 4 5

Heel Cup: Shallow Medium Deep **Morton's Extension:** Right Left

6. Once all of the required fields have been entered, the “Save” icon will appear.

iPad 3:54 PM 83%

[Main Menu](#) Patient Form

5050 South 2nd Street
Milwaukee, WI 53207
1-844-637-4637
www.anodyneshoes.com

ANODYNE

Account No: 12345
Account Name: Anodyne
Date: 11/22/16
P.O. No: _____

Patient Name: **Test** Height: _____ Weight: _____

Activity Level: Low Moderate High Shoe Order Below Inserts Only Order

Insert Quantity (pairs): 1 2 3 Toe Filler: Left Right

Men's: Size: 7.5 8 8.5 9 9.5 10 10.5 11 11.5 12 13 14
Width: M W XW

Women's: Size: 5.5 6 6.5 7 7.5 8 8.5 9 9.5 10 11 12
Width: M W XW


Arch Height: Same as scan
Lower than scan
Higher than scan

Off-Load Metatarsal Heads:
Right: 1 2 3 4 5
Left: 1 2 3 4 5


Heel Cup: Shallow Medium Deep **Morton's Extension:** Right Left

Medial Arch Flange: Right Left


- Click the “Save” icon, and then you will see a pop-up notification that your order form has been saved. Select “OK” and move on to the next step.

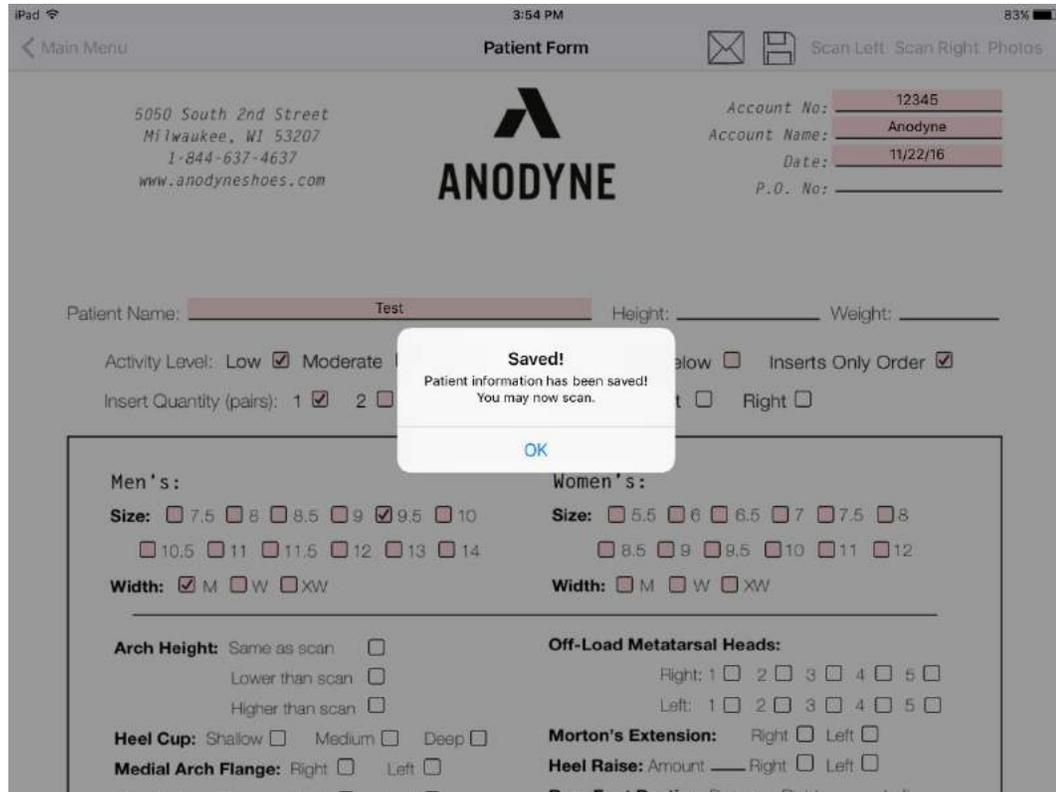


- Now you will see that the “Scan Left” and “Scan Right” options are available in the top right corner. Select whichever option you would like to do first.

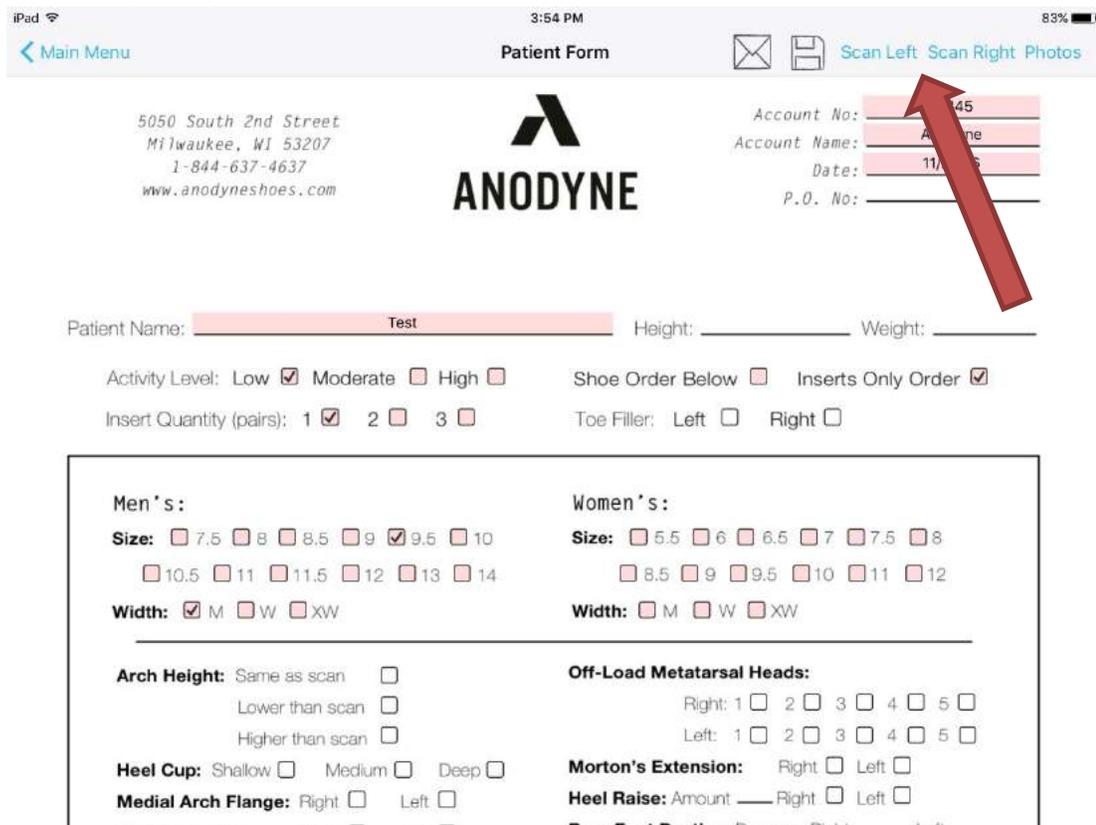

9. With the patient properly positioned, you're ready to begin your scan. Once the scanner is mapping the red/orange/yellow to the bottom of the foot, click the "Scan" button. Zooming in or out by pinching or spreading your fingers will allow you to change the size of the 3D box. You'll have to do this, in addition to moving closer or further from the patient in order to optimize the mapping.

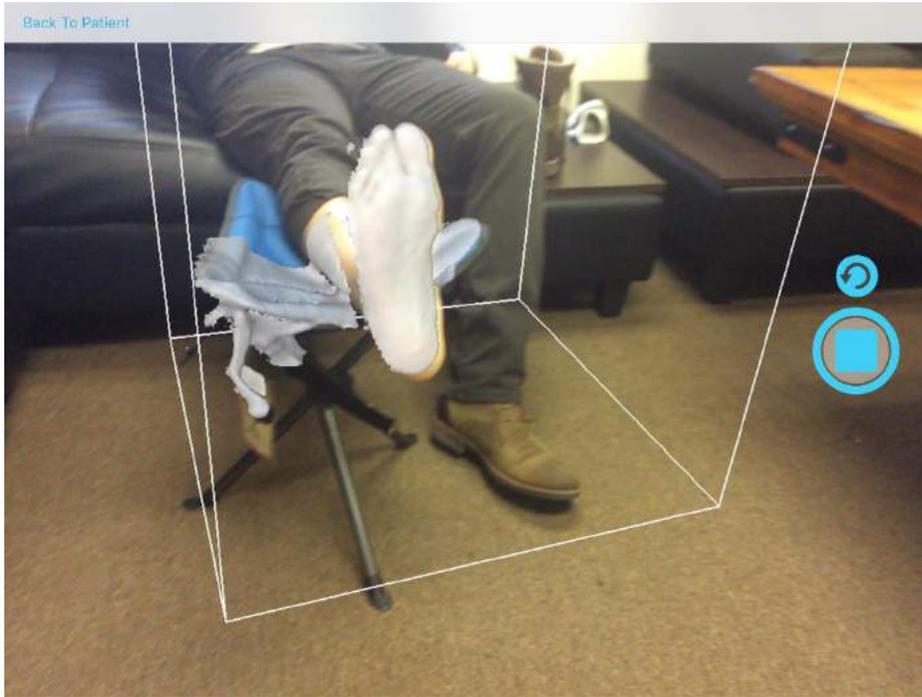


10. After you select "Scan," the 3D image will begin rendering. Slowly move the camera around the foot to make sure that all of the details are picked up. There will be an alert if you're moving the camera too quickly or not letting the 3D image render completely.

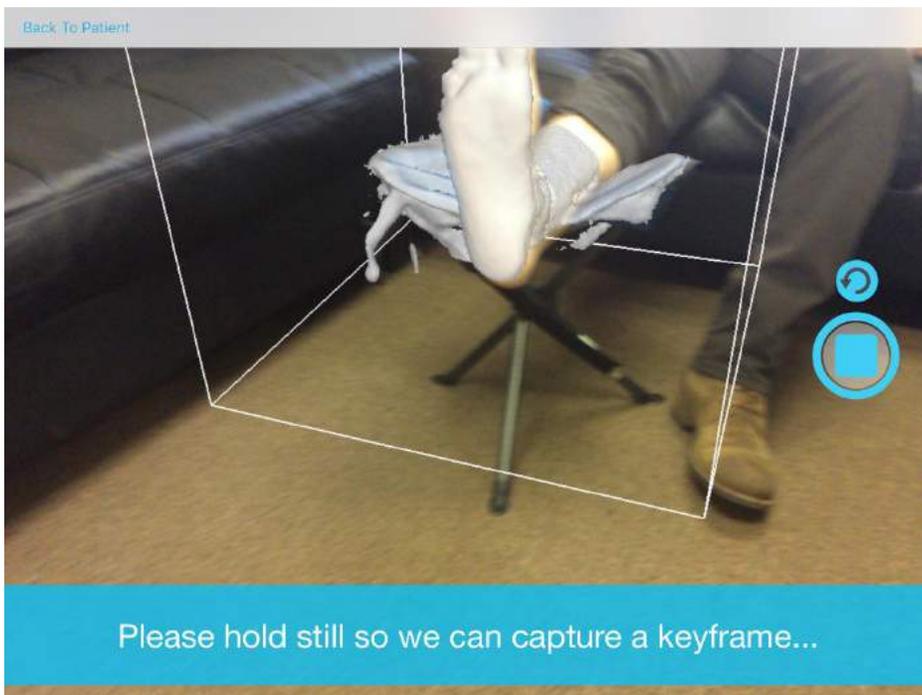

11. Once you're satisfied with the scan, click the "Stop" icon.



12. In the "Scan Result Viewer" you can inspect the scan by moving the 3D image around. Select "Rescan," located in the top left corner, if you are unhappy with the image. Otherwise, click the "Save" icon, in the top right corner, to proceed.

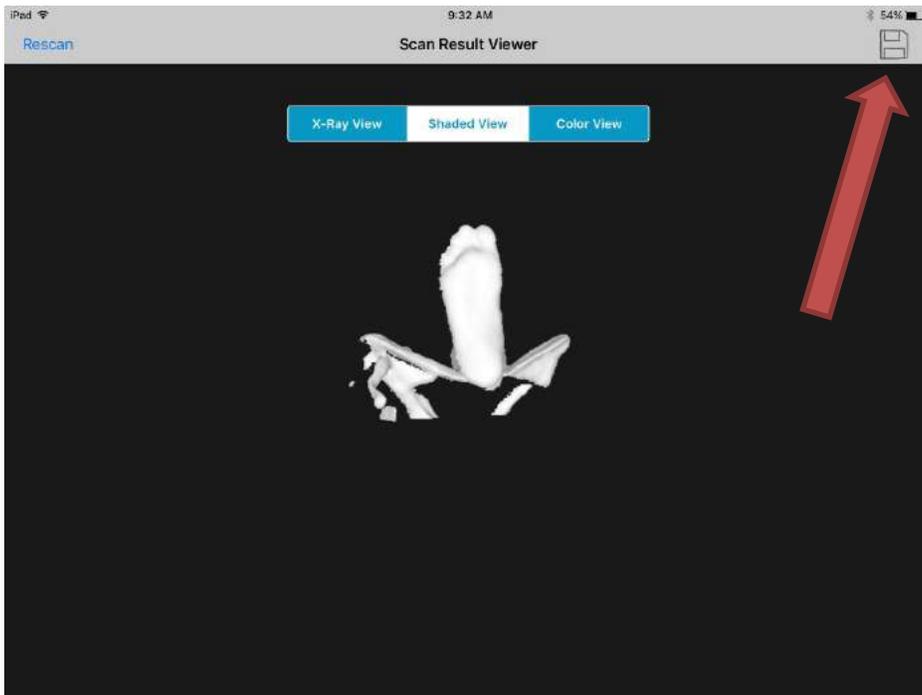

13. You'll then be taken back to the "Patient Form" where you will now select to scan the other foot and repeat steps 6 through 8.

5050 South 2nd Street
Milwaukee, WI 53207
1-844-637-4637
www.anodyneshoes.com

ANODYNE

Account No: 12345
Account Name: Anodyne
Date: 11/22/16
P.O. No:

Patient Name: Test Height: _____ Weight: _____

Activity Level: Low Moderate High Shoe Order Below Inserts Only Order

Insert Quantity (pairs): 1 2 3 Toe Filler: Left Right

Men's:
Size: 7.5 8 8.5 9 9.5 10
 10.5 11 11.5 12 13 14
Width: M W XW

Women's:
Size: 5.5 6 6.5 7 7.5 8
 8.5 9 9.5 10 11 12
Width: M W XW

Arch Height: Same as scan
Lower than scan
Higher than scan

Heel Cup: Shallow Medium Deep

Medial Arch Flange: Right Left

Off-Load Metatarsal Heads:
Right: 1 2 3 4 5
Left: 1 2 3 4 5

Morton's Extension: Right Left

Heel Raise: Amount _____ Right Left

14. In addition to taking the scans, our App also gives you the option of sending us photos of the patient's feet. To add Photos, click the "Photos" text in the top right of the screen.


*If no special accommodations are requested, inserts will be made based off of scan.

Men's			Women's		
Style No.	Description	Color	Style No.	Description	Color
<input type="checkbox"/> 22	Sport Runner	Black	<input type="checkbox"/> 19	Casual Mary Jane	Black
<input type="checkbox"/> 22	Sport Runner	Grey	<input type="checkbox"/> 19	Casual Mary Jane	Cognac
<input type="checkbox"/> 22	Sport Runner	Blue	<input type="checkbox"/> 23	Sport Runner	Black
<input type="checkbox"/> 30	Casual Dress	Black	<input type="checkbox"/> 23	Sport Runner	Grey
<input type="checkbox"/> 30	Casual Dress	Whiskey	<input type="checkbox"/> 23	Sport Runner	Teal
<input type="checkbox"/> 38	Sport Walker	White	<input type="checkbox"/> 31	Sport Walker	White
<input type="checkbox"/> 38	Sport Walker	Black	<input type="checkbox"/> 31	Sport Walker	Black
<input type="checkbox"/> 44	Trail Walker	Dark Grey	<input type="checkbox"/> 33	Casual Mary Jane	Black Stretch
<input type="checkbox"/> 44	Trail Walker	Stone	<input type="checkbox"/> 45	Sport Jogger	Black
<input type="checkbox"/> 46	Sport Jogger	Black	<input type="checkbox"/> 45	Sport Jogger	Grey
<input checked="" type="checkbox"/> 46	Sport Jogger	Grey	<input type="checkbox"/> 45	Sport Jogger	Purple
<input type="checkbox"/> 46	Sport Jogger	Blue	<input type="checkbox"/> 49	Trail Walker	Dark Grey
<input type="checkbox"/> 52	Casual Dress	Black	<input type="checkbox"/> 49	Trail Walker	Stone
<input type="checkbox"/> 52	Casual Dress	Whiskey	<input type="checkbox"/> 51	Casual Dress	Black
<input type="checkbox"/> 56	Trail Boot	Oil Black	<input type="checkbox"/> 51	Casual Dress	Cognac
<input type="checkbox"/> 56	Trail Boot	Whiskey	<input type="checkbox"/> 55	Trail Boot	Oil Black
<input type="checkbox"/> 64	Casual Comfort	Black	<input type="checkbox"/> 55	Trail Boot	Whiskey
<input type="checkbox"/> 64	Casual Comfort	Whiskey	<input type="checkbox"/> 63	Casual Comfort	Black Stretch
<input type="checkbox"/> 66	Casual Comfort	Black Stretch	<input type="checkbox"/> 67	Casual Comfort	Chocolate
<input type="checkbox"/> 72	Casual Sport	Oil Black	<input type="checkbox"/> 67	Casual Comfort	Black
<input type="checkbox"/> 72	Casual Sport	Oil Brown	<input type="checkbox"/> 75	Casual Sport	Black
<input type="checkbox"/> 74	DD - Sport Jogger	Black	<input type="checkbox"/> 75	Casual Sport	Saddle

15. Next, select the “+” to add a new photo.




16. You can choose to use a previously saved picture (click “Camera Roll”), or take a completely new one (click “Camera”).

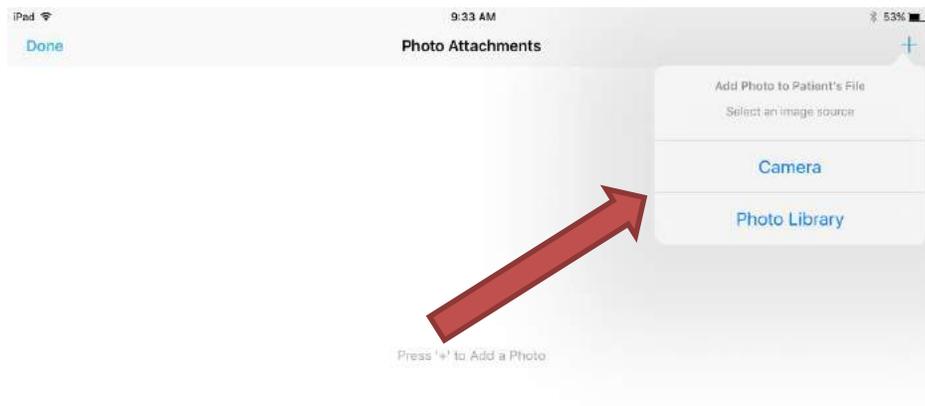

17. Once you've selected or taken your photo you will have the ability to use your finger or stylus to draw on it. This helps to highlight any sort of specific complications, wounds, and/or call-outs. Click "Save."



18. Once you're satisfied with the picture(s) that you've added, select "Done" in the top left corner of the screen, and you will be taken back to the "Patient Form."

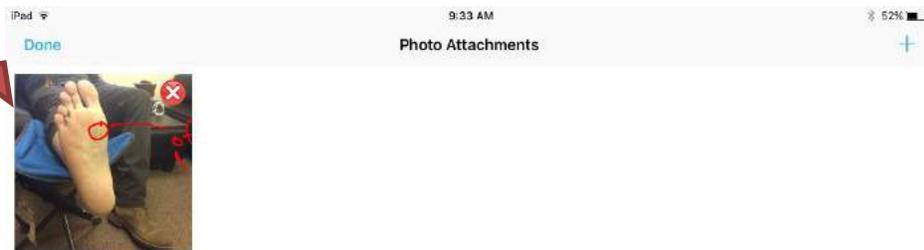

19. After both feet have been successfully scanned, the order form has been filled out, and/or pictures have been added, you're ready to send everything over to us. Review all of the information one last time, and click the "Email" icon to send the files.

5050 South 2nd Street
Milwaukee, WI 53207
1-844-637-4637
www.anodyneshoes.com

ANODYNE

Account No: 12345
Account Name: Anodyne
Date: 11/22/16
P.O. No: _____

Patient Name: Test Height: _____ Weight: _____

Activity Level: Low Moderate High Shoe Order Below Inserts Only Order

Insert Quantity (pairs): 1 2 3 Toe Filler: Left Right

Men's:
Size: 7.5 8 8.5 9 9.5 10
 10.5 11 11.5 12 13 14
Width: M W XW

Women's:
Size: 5.5 6 6.5 7 7.5 8
 8.5 9 9.5 10 11 12
Width: M W XW

Arch Height: Same as scan
Lower than scan
Higher than scan

Heel Cup: Shallow Medium Deep

Medial Arch Flange: Right Left

Off-Load Metatarsal Heads:
Right: 1 2 3 4 5
Left: 1 2 3 4 5

Morton's Extension: Right Left

Heel Raise: Amount _____ Right Left

20. After you click the "Email" icon, you'll receive a pop-up notifying you that your order was sent successfully.

5050 South 2nd Street
Milwaukee, WI 53207
1-844-637-4637
www.anodyneshoes.com

ANODYNE

Account No: 12345
Account Name: Anodyne
Date: 11/22/16
P.O. No: _____

Patient Name: Test Height: _____ Weight: _____

Activity Level: Low Moderate High Shoe Order Below Inserts Only Order

Insert Quantity (pairs): 1 2 3 Toe Filler: Left Right

Men's:
Size: 7.5 8 8.5 9 9.5 10
 10.5 11 11.5 12 13 14
Width: M W XW

Women's:
Size: 5.5 6 6.5 7 7.5 8
 8.5 9 9.5 10 11 12
Width: M W XW

Arch Height: Same as scan
Lower than scan
Higher than scan

Heel Cup: Shallow Medium Deep

Medial Arch Flange: Right Left


Off-Load Metatarsal Heads:
Right: 1 2 3 4 5
Left: 1 2 3 4 5

Morton's Extension: Right Left

Heel Raise: Amount _____ Right Left

Order On The Way...
Your order is being processed! You will get a confirmation shortly!

21. You will also receive an email confirming that the scan was sent, with a PDF attachment of the order form for your records.

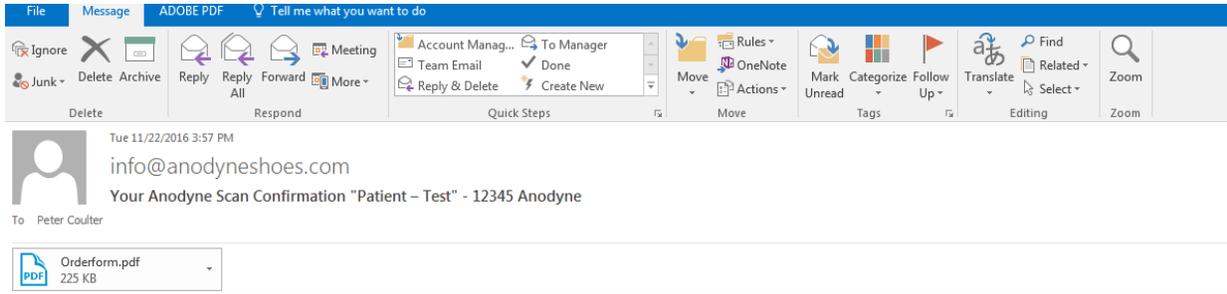


Account #: 12345
Account Name: Anodyne

Hi Peter Coulter,

We just received the scan that you sent for Patient - Test. Attached is a copy of the PDF order form. If you have any questions, comments, or concerns about this order, please don't hesitate to reach out to us at 1-844-637-4637 or info@anodyneshoes.com.

We appreciate your business!

22. If you would like to revisit or check scans/orders for previous patients, select the "Saved Patients" option on the home screen.

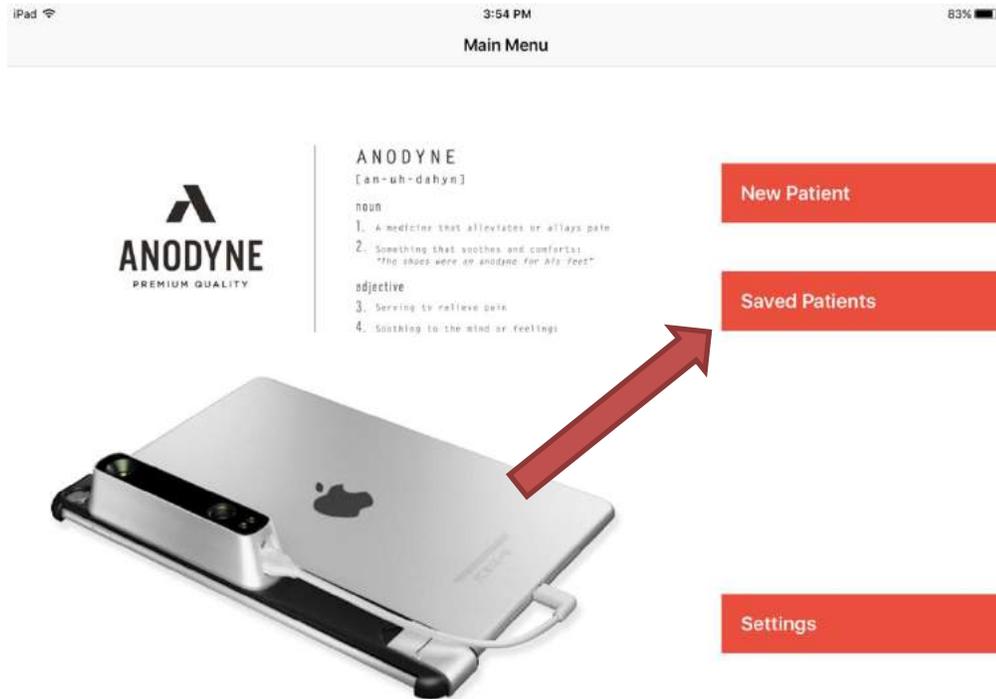

23. Here you can see when previous orders were sent.



24. By clicking into the patient, you can review the actual order form and scans, whenever you need to.

